



## STEP ONE (Home Screen)

Turn dial to "Setting" and select to enter.

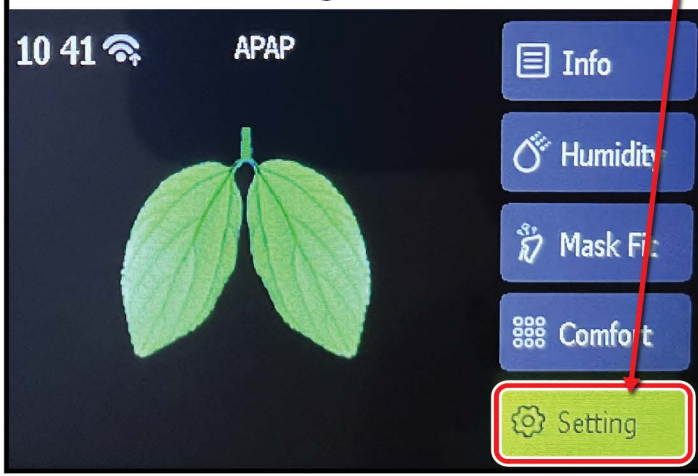


## STEP FOUR

Turn the dial to highlight your home Wi-Fi network and click to select.

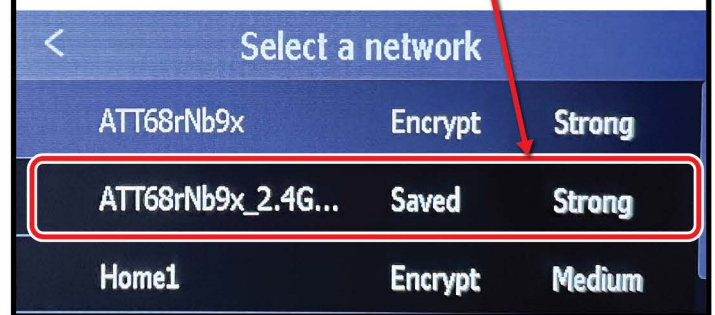


## STEP FIVE

Enter your Wi-Fi password (if needed), then click "Join."

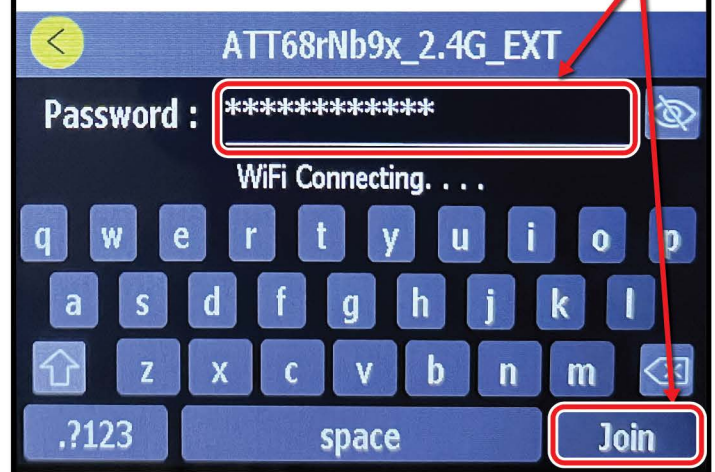

## STEP TWO

Turn dial to "Wi-Fi" and select to enter.

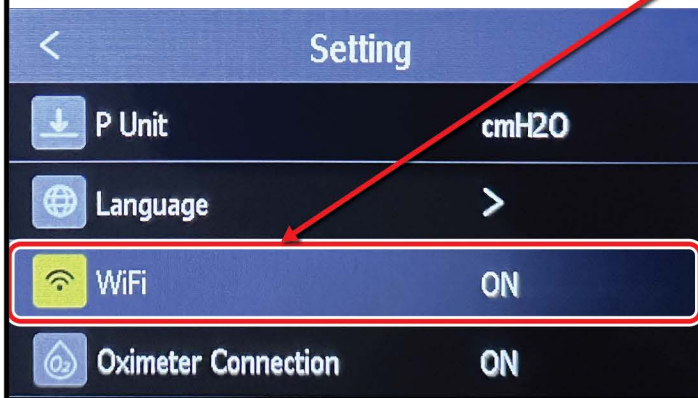

## STEP THREE

Toggle Wi-Fi on (if not already on), then turn dial to "Search" and click to search all Wi-Fi networks available.



## STEP SIX

Device confirms connection to your Wi-Fi, and your usage information is transmitted daily to ResAssist (compliance platform).

